

Opiates, Serum or Plasma, Quantitative

Order Name: **OPIATE S/P**

Test Number: 4312130

Revision Date: 10/07/2020

TEST NAME	METHODOLOGY	LOINC CODE
Hydrocodone, S/P, Quant	Quant Liquid Chromatography-Tandem Mass Spectrometry	3680-6
Hydromorphone, S/P, Quant	Quant Liquid Chromatography-Tandem Mass Spectrometry	3683-0
Codeine, S/P, Quant	Quant Liquid Chromatography-Tandem Mass Spectrometry	3506-3
Morphine, S/P, Quant	Quant Liquid Chromatography-Tandem Mass Spectrometry	3827-3
6-acetylmorphine, S/P, Quant	Quant Liquid Chromatography-Tandem Mass Spectrometry	12788-6
Oxycodone, S/P, Quant	Quant Liquid Chromatography-Tandem Mass Spectrometry	3893-5
Oxymorphone, S/P, Quant	Quant Liquid Chromatography-Tandem Mass Spectrometry	60467-8

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (mL)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL (0.5 mL)	Serum	Clot Activator (Red Top, No-Gel)	Refrigerated
Alternate 1	1 mL (0.5 mL)	Plasma	Sodium fluoride (gray top)	Refrigerated
Alternate 2	1 mL (0.5 mL)	Plasma	EDTA (Lavender Top)	Refrigerated
Alternate 3	1 mL (0.5 mL)	Plasma	Sodium Heparin (Green Top / No-Gel)	Refrigerated
Instructions	Separate serum or plasma from cells ASAP or within 2 hours of collection. Transfer 1 mL serum or plasma to a Standard Transport Tube. Unacceptable Conditions: Hemolyzed specimens. Separator tubes. Specimens exposed to repeated freeze/thaw cycles. Plasma or whole blood collected in light blue (sodium citrate). Stability: After separation from cells: Ambient: 1 week; Refrigerated: 2 weeks; Frozen: 3 years			

GENERAL INFORMATION	
Testing Schedule	Mon, Wed, Fri
Expected TAT	2-5 Days
Notes	Reference Lab: ARUP Test Code: 92354 Click Here to view information on the ARUP website.
CPT Code(s)	80364; (Alt code: G6056)
Lab Section	Reference Lab