## Labcorp Oklahoma, Inc. Test Directory

## **Eosinophil Cationic Protein**

Order Name: **Eos Cation Prot** 

Test Number: 6906929 Revision Date: 12/12/2022

TEST NAME		METHO	ODOLOGY	LOINC CODE
Eosinophil Cationic Protein		Immur	noCAP	25638-8
SPECIMEN REQUIREM	MENTS			
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL (0.25 mL)	Serum	Clot Activator SST	Refrigerated or Frozen
Instructions	Notes: 0.3 mL (Note: This volume Does NOT allow for repeat testing)  Specimen Type: NMR lipoprofile serum-separator tube (LabCorp item ID 60360). Other tubes should not be used as they may produce erroneous results.  Specimen Storage: Refrigerate  Specimen Collection: Allow the sample to stand for 60 to 120 minutes to release the ECP in the cells prior to centrifugation. Separate the serum after 60 to 120 minutes and transfer to a plastic transport tube.  Specimen Stability: Ambient: 5 days, Refrigerated: 14 days, Frozen: 14 days			

GENERAL INFORMATION		
Expected TAT	4 - 7 days	
Performing Labcorp Test Code	004180	
Notes	Labcorp Test Code: 004180	
CPT Code(s)	83520	
Lab Section	Reference Lab	

Service provided by Labcorp Oklahoma, Inc. All Rights Reserved. © 2003 - 2026